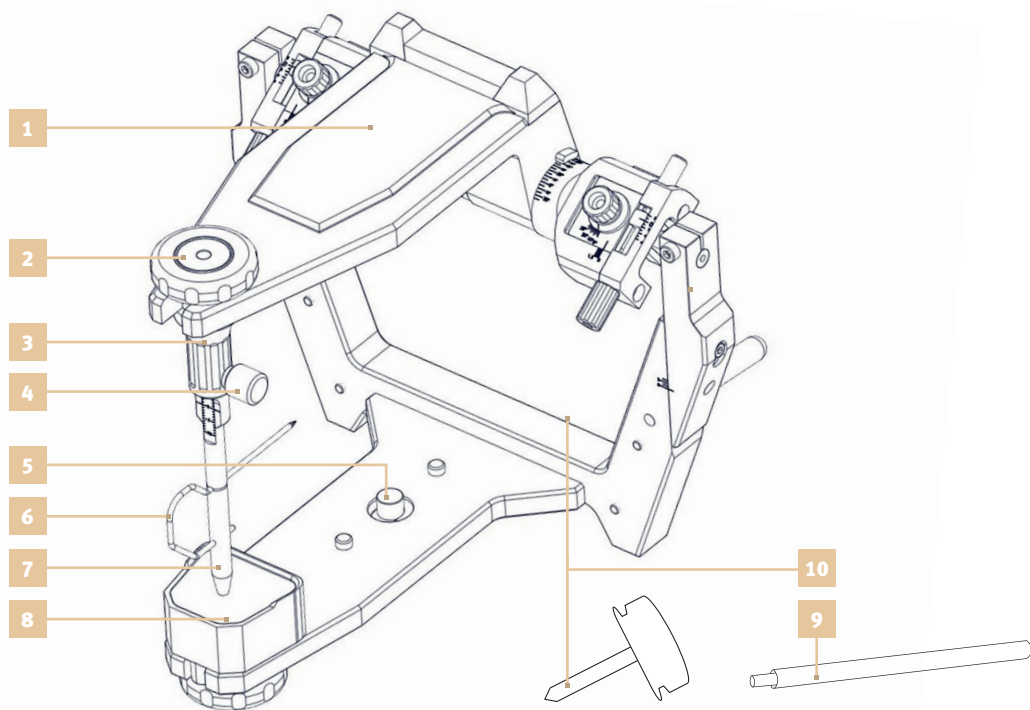
-  *Gebrauchsanleitung*  *Instruction manual*  *Mode d'emploi*
-  *Istruzioni per l'uso*  *Instrucciones de uso*  *Instrukcja użytkowania*
-  *Руководство по эксплуатации*

|                                                                                  |                                                                                       |              |
|----------------------------------------------------------------------------------|---------------------------------------------------------------------------------------|--------------|
|   | Gebrauchsanleitung.....                                                               | Seite 6/7    |
|                                                                                  | <b>Achtung: Vor der Einarbeitung die Vollständigkeit der Komponenten überprüfen</b>   |              |
|  | Instruction manual.....                                                               | Page 10/11   |
|                                                                                  | <b>Attention: Verify the completeness of the components before the training</b>       |              |
|  | Mode d'emploi.....                                                                    | Page 14/15   |
|                                                                                  | <b>Attention : Avant de commencer, vérifiez que les composants sont au complet.</b>   |              |
|  | Istruzioni per l'uso .....                                                            | Pagina 18/19 |
|                                                                                  | <b>Attenzione: prima dell'incorporazione verificare la completezza dei componenti</b> |              |
|  | Instrucciones de uso .....                                                            | Página 22/23 |
|                                                                                  | <b>Atención: antes del montaje, comprobar que los componentes estén completos</b>     |              |
|  | Instrukcja użytkowania .....                                                          | Strona 26/27 |
|                                                                                  | <b>Uwaga: Przed wdrożeniem sprawdzić kompletność komponentów</b>                      |              |
|  | Руководство по эксплуатации.....                                                      | Стр. 30/31   |
|                                                                                  | <b>Внимание: Перед приработкой проверьте компоненты, входящие в состав комплекта</b>  |              |

## CORSOART® Artikulatoren

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## Accessories-Articles for reorder

|   |                                                                            |                  |    |                                                                   |                |
|---|----------------------------------------------------------------------------|------------------|----|-------------------------------------------------------------------|----------------|
| 1 | Removable plate with magnetic closure for model <b>CORSOART®</b> BKR, red  | Art-No.: 12167-M | 6  | Incisalneedle for <b>CORSOART®</b> articulators                   | Art-No.: 12408 |
|   | Removable plate with magnetic closure for model <b>CORSOART®</b> BKR, blue | Art-No.: 12267-M | 7  | Anterior tooth guide pin for <b>CORSOART®</b> A-Line              | Art-No.: 12141 |
|   | Removable plate with magnetic closure for model <b>CORSOART®</b> BKR, gold | Art-No.: 12367-M |    | Anterior tooth guide pin for <b>CORSOART®</b> AC-Line             | Art-No.: 12241 |
| 2 | Mounting screw for anterior tooth guide pin-Plate & pin holder             | Art-No.: 12406   |    | Anterior tooth guide pin for <b>CORSOART®</b> S-Line              | Art-No.: 12321 |
| 3 | Anterior tooth guide pin holder for <b>CORSOART®</b> A-Line, red           | Art-No.: 12142   | 8  | Plate plain 0° for <b>CORSOART®</b> A-Line & Artex®, red          | Art-No.: 12143 |
|   | Anterior tooth guide pin holder for <b>CORSOART®</b> AC-Line, blue         | Art-No.: 12242   |    | Plate plain 0° for <b>CORSOART®</b> AC-Line & Artex® Carbon, blue | Art-No.: 12243 |
|   | Anterior tooth guide pin holder for <b>CORSOART®</b> S-Line, gold          | Art-No.: 12322   |    | Plate plain 0° for <b>CORSOART®</b> S-Line & SAM®, gold           | Art-No.: 12323 |
| 4 | Anterior tooth guide pin screw                                             | Art-No.: 12407   | 9  | Support pin, screwable for <b>CORSOART®</b> A-, AC- and S-Line    | Art-No.: 12401 |
| 5 | Mounting screw-kit for <b>CORSOART®</b> A-Line                             | Art-No.: 12402   | 10 | Tool key for <b>CORSOART®</b> BKR articulators                    | Art-No.: 12421 |
|   | Mounting screw-kit for <b>CORSOART®</b> AC-Line                            | Art-No.: 12403   |    |                                                                   |                |
|   | Mounting screw-kit for <b>CORSOART®</b> S-Line                             | Art-No.: 12404   |    |                                                                   |                |

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
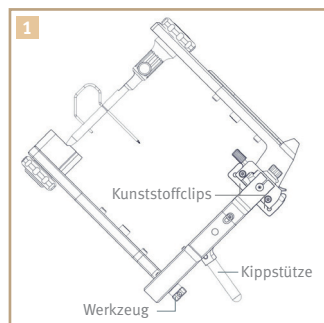
# Instruction manual CORSOART® articulators


## 45° tilt support/tool

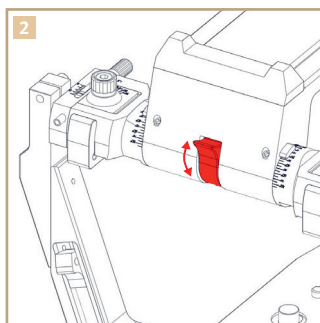The tilt supports are located at the back of the articulator. This lets the articulator be tilted back to less than 45°. The tool for the following functions is located in the base of the articulator frame.


## Opening/closing the centric

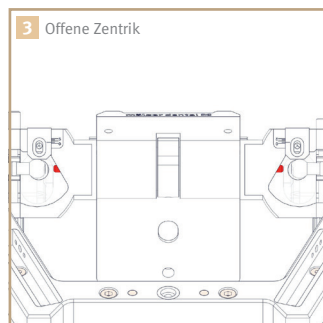To open the centric, move the swivel arm up as far as it will go – this releases the centric. To close the centric, move the swivel arm in the opposite direction – this will again fix the condyles in place in a central position.

## Opening the centric

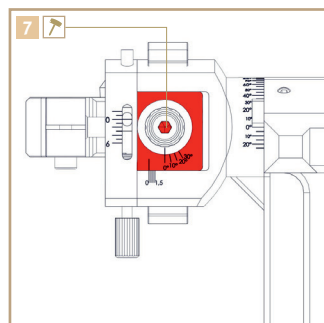
The condyles are no longer fixed and the centric is released. The top half of the articulator can be moved freely.

## Immediate side shift (ISS)


Loosen the locking screw and set the ISS to the desired position.

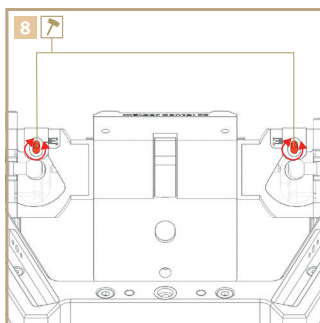Can be set from 0 mm to 1.5 mm per side

## Setting protrusion/retrusion

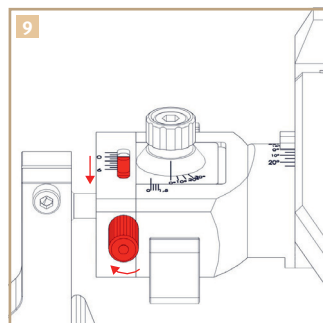To change either of these settings, first loosen the locking screw on the block underneath the top half of the articulator.

## Setting protrusion

Protrusion can be set after loosening the locking screw.

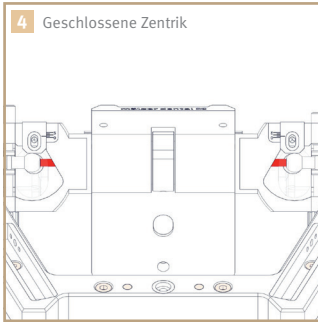
Can be set from 0 mm to 6 mm

## General information & care instructions

Precision instruments such as articulators must comply with the highest requirements. Therefore, the following instructions for use of these devices should be strictly observed.


- Regular inspection of the axis-ball-system and the condylar path for contamination. This secures the accurate functionality of the articulator and maintains the smooth running of the ball on the axis.
- Contamination that took place here, can be removed by evaporation. Subsequently blow dry remaining moisture.
- Under no circumstances may the articulator be put in an extraction unit, pressure pot or ultrasonic bath.

- Use only cleaning agents that are suitable for the cleaning of light metals and apply according to the manufacturer's instructions. Cleaning agents, which have a pH value of below 5 and above 8, may not be used, in order to prevent damage to the anodised surfaces.
- Please, do not use a plaster dissolution, in order to prevent damage to the anodised surfaces.
- For cleaning and care, please, use only the special care spray (Art.-No. 12015).

#### Closing the centric

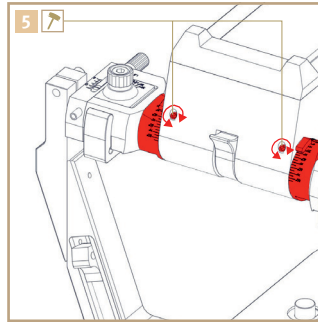
The condyles are again fixed in place, the centric is visibly closed.

#### Condylar path inclination

Loosen the two HCl set screws with the centric closed and set the angle for the condylar path inclination on the condylar housing. Re-tighten the HCl set screws after setting the angle.

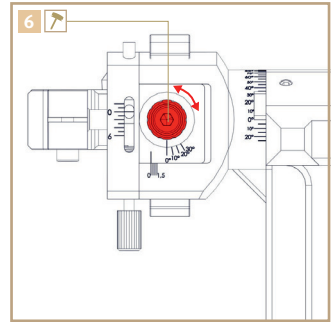
Angle can be set from  $-25^{\circ}$  to  $+80^{\circ}$

#### Setting the Bennett angle

Loosen the locking screw and set the Bennett angle to the desired position.

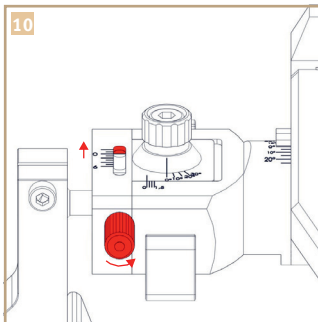
Angle can be set from  $0^{\circ}$  to  $+30^{\circ}$

#### Setting retrusion

Retrusion can be set after loosening the locking screw.

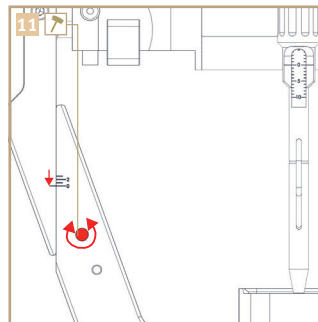
Can be set from 0 mm to 2 mm

#### Setting distraction

Distraction can be set after loosening the locking screw on the side of the articulator.

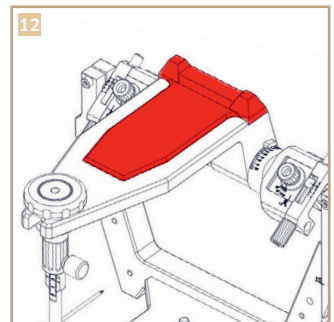Can be set from 0 mm to 3 mm

#### Removable plates

The removable plate is secured with a magnetic closure and can be removed simply by lifting up and out.

### Legal information

All information in this documentation has been prepared with great care and is subject to changes and mistakes. We assume no legal responsibility on consequences of incorrect information or improper use.

#### Tool required

The special tool is located at the back of the bottom half of the articulator.

# Instruction manual CORSOART® articulators

## Contraindications

The articulator should not be used in patients who have allergic reactions to the material used in the medical device.

## Target Groups

This document is intended for dentists and dental laboratory technicians as users of the articulator, and for the staff in the dental practice who disinfect and clean the articulator.

## Transport and Storage

### Information on the Packaging

**Note:** Please retain the packaging if you wish to ship the device out for service or repair.

The externally printed symbols are for transport and storage and have the following meaning:

|                                                                                   |                                                                                                                                                                  |
|-----------------------------------------------------------------------------------|------------------------------------------------------------------------------------------------------------------------------------------------------------------|
|  | Symbol for „Read instructions for use“                                                                                                                           |
|  | Symbol for „Serial Number“                                                                                                                                       |
|  | Symbol for „Article Number“                                                                                                                                      |
|  | Symbol for „Date of manufacture“                                                                                                                                 |
|  | Symbol for „Manufacturer and manufacturer address“                                                                                                               |
|  | CE mark (Communauté Européenne). A product bearing this mark complies with the requirements of the relevant EU directive, i.e. the applicable European standard. |

## Nameplate

The nameplate is placed on the packaging of the articulator.

**CORSOART® Articulator Typ BKR**

Mälzer Dental e.K.  
Owner: Carsten Althaus  
Schlesienweg 27  
31515 Wunstorf, GERMANY

Phone: +49 (0) 5033/9630990  
Fax: +49 (0) 5033/963099-99  
E-Mail: [info@maelzer-dental.de](mailto:info@maelzer-dental.de)  
Web: [www.maelzer-dental.de](http://www.maelzer-dental.de)

**mälzer dental** 

### General intended purpose - proper use

The user must be aware of the functional safety and the proper condition of the device before each use of the device.

The articulator is a medical device according to the applicable national legislation. The articulator is designed so that no danger to the user and/or patient shall arise.

This product is only designed for use in dental practices. Any other use is not permitted.

Proper use also involves observing all the stipulations in the instructions for use and adhering to the disinfection and cleaning instructions.

The general guidelines and/or national laws and regulations applicable to medical devices must be applied to the product and fulfilled in accordance with its intended use.

It is the obligation of the user:

- to only use tools that are not defective.
- to protect the patient and third parties from any hazards.
- to avoid contamination by the product.

During use, national legal regulations are to be followed, and most particularly:

- The applicable health and safety regulations.
- The applicable accident prevention measures.

In Germany, operators, those responsible for devices, and users are obliged to operate their devices in compliance with the regulations of the German Medical Devices Act (MPG).

The user is responsible for cleaning and disinfecting the articulator.

### Intended use

Articulators are devices for simulating the temporomandibular joint movement. For this purpose, plaster models of the dental arches of the upper and lower jaw are mounted in occlusion in the articulator. Subsequently, the movements of the jaws with respect to one another can be simulated, which is essential for the preparation of replacement teeth, partial and full prosthetics, or splints.

Depending on the setting options for the articulator, mastication movements can be precisely simulated to varying extents. This makes it possible to produce fixed tooth replacements, such as crowns and bridges, or removable tooth replacements, such as partial prosthetics, full dentures or splints.

The following types of articulator can be distinguished:

- Non-adjustable articulators
- Semi-adjustable articulators
- Fully adjustable articulators

The various models differ with regard to their setting options, whether it be with regard to individual adjustment of the Bennett angle, the condylar track inclination, the terminal hinge axis, or the incisal guidance. The incisal disc is exchangeable so that the gliding angle can also be individually adjusted. With lateral movements of the mandible, condylar displacement occurs. A distinction is made between an immediate side shift and a progressive side shift. [Reitemeier, 2006]

### Arcon Articulators

Furthermore, a distinction can also be made between so-called non-arcon and arcon articulators. In the former, the structure of the device is the reverse of the human jaw (the condyles are mounted in the upper part of the articulator), while in the latter, the structure corresponds to the human anatomical conditions of the temporomandibular joint. This has no bearing on the function, however, since the relative movement of the lower jaw to the upper jaw is the same for both technical approaches. In non-arcon articulators the course of movement in the articulator joint is reversed compared to the natural process so that it requires some getting used to. [Seeher, 2004]

### Intended medical use

The medical device is designed for modelling a patient's real and individual physiological or non-physiological temporomandibular joint movements, as may be required when implementing splints, mock-ups, functional crowns and bridges, partial and total prosthetics, and treatments for craniomandibular dysfunction.

### Other intended use

Articulator for the two- and three-dimensional simulation of mandibular movement and temporomandibular function.

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